


Infinity Feeding Pump

CLEANING

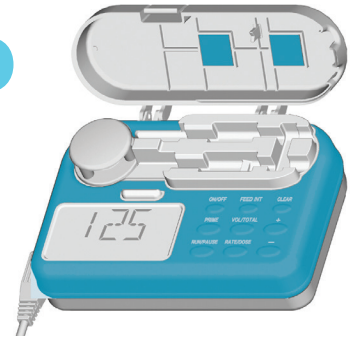

WARNING: To avoid electrical shock, never clean the pump or Infinity AC Adapter/Charger, while the charger is plugged into an outlet, or the pump is turned on.

WARNING: Make sure the Infinity AC Adapter/Charger is completely dry before plugging it into an electrical outlet.

To Clean the Infinity Enteral Feeding Pump:

Pump may be cleaned with warm soapy water (standard dish soap) and a nonabrasive sponge or soft cloth. Use a cotton swab to clean pathways of cassette receptacle and to remove teal silicon residue from pump wheel roller pins.

OR - Apply one of the following solutions for approximately 10 minutes, then wipe pump clean with a damp cloth or sponge:

- » 5% bleach and water solution
- » Multipurpose household disinfectant cleaner

NOTE: Clean pump as needed. If the pump is used on more than one patient, clean between each patient.

NOTE: Avoid harsh cleaners/disinfectants. They may cause damage to pump surfaces and may affect pump functionality. Moog has tested the following disinfectants for acceptability for cleaning/disinfecting the Infinity pump. These cleaning products are categorized below:

Acceptable	Unacceptable
Metrex CaviCide	Ball Ruthless
5% Bleach and Water Solution	Wex-Cide 128
Warm Soapy Water (Standard Dish Soap)	Alcohol
	Vesphene Ilse

To Clean Sentido Backpacks:

The Sentido feeding pump backpacks should only be hand washed or dry cleaned.

To Clean the AC Adapter/Charger:

Infinity AC Adapter/Charger normally does not require cleaning. When desired, a dry or slightly damp cloth may be used to clean the outside surface of AC Adapter/Charger while it is disconnected from the wall outlet.

